PLEASE CONTACT MOEN FIRST

For Installation Help, Missing or Replacement Parts

(USA)
1-800-BUY-MOEN (1-800-289-6636)
WWW.MOEN.COM
(Canada)
1-800-465-6130
WWW.MOEN.CA

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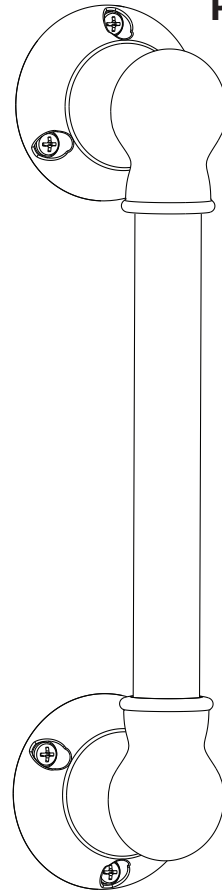
Installation Instructions For Hand Grips with Press and Mark

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Thank you for purchasing this Moen® product. Moen is a leader in grab bars, offering top-quality products that are attractive and innovative, with performance features like a 250 lb. weight pull capacity. The entire Moen family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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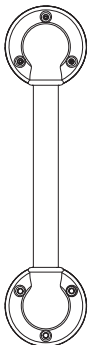
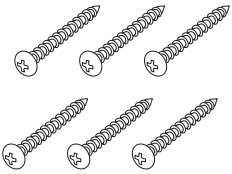
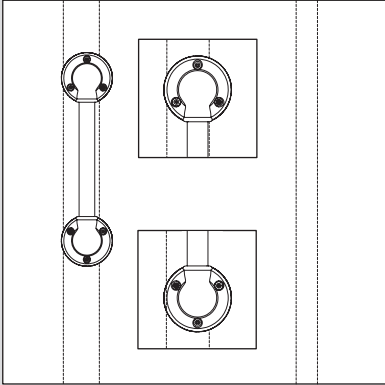
To support 250 lbs, you need a minimum of 2 screws per post to be in a stud.

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IMPORTANT:

Please note that screws **MUST** be mounted into a wood stud for secure installation.

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1

Position your hand grip vertically and remember that **BOTH** ends of the hand grip **MUST** be positioned over a wood stud.

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2

Locate the stud behind the wall by using a stud finder or other stud locating method. Locate the edges of the stud. To support 250 lbs, you need a minimum of 2 screws per post to be in a stud.

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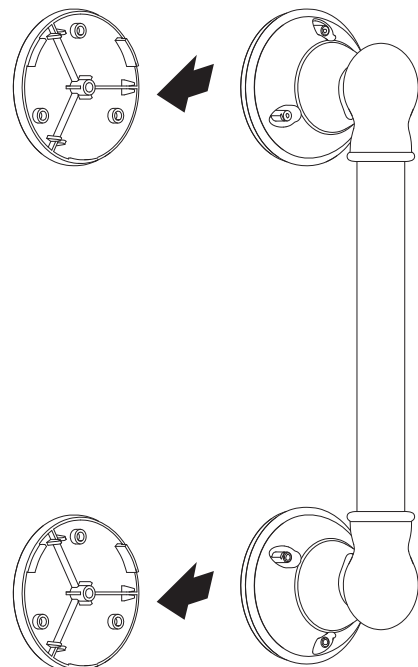
5

Press & Mark

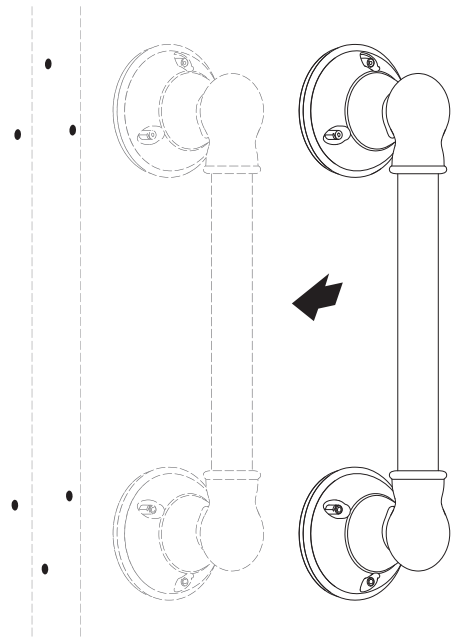
Align hand grip in desired location, press & mark by pushing both posts firmly and evenly against the wall. Hold for a few seconds, then set assembled product aside. You'll now see marks showing you exactly where to drill.

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6

Verify

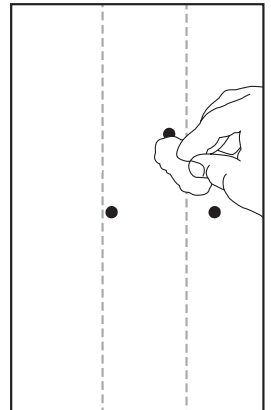
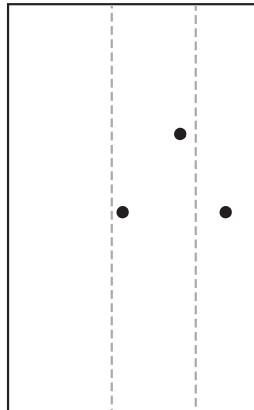
If you don't like the marking on the wall, wipe the wall clean using a damp cloth and re-mark the location.

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

7

Remove twist ties (x2) and ink stampers (x2) from the product.

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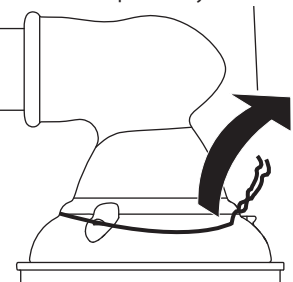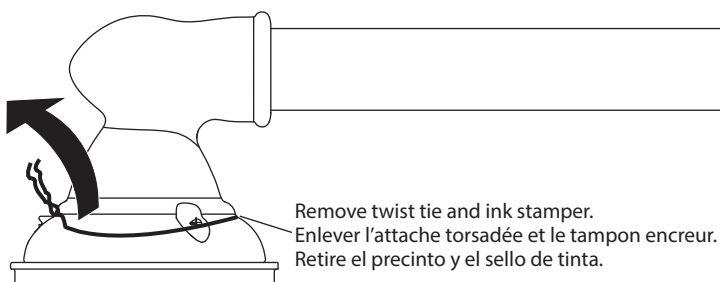8

Drill 1/8" diameter pilot holes at each screw location. Make sure to drill into the stud. Refer to Figure 3, Drill Location diagram on page 3 for hole locations.

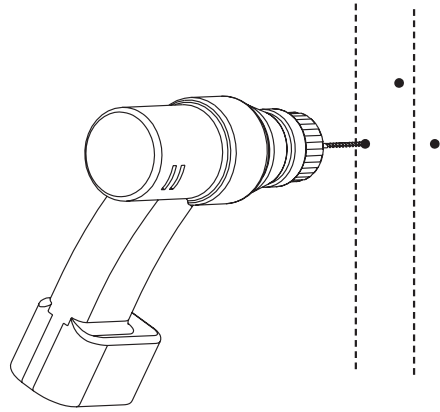
NOTE: For easier screw installation, drill pilot hole the length of the screw.

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
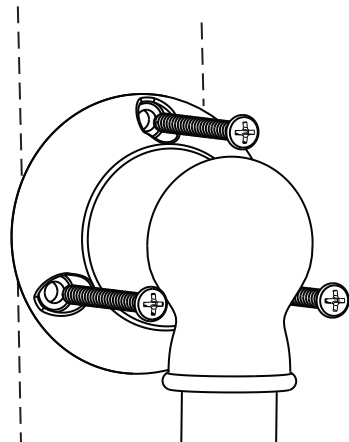
Installation is complete.

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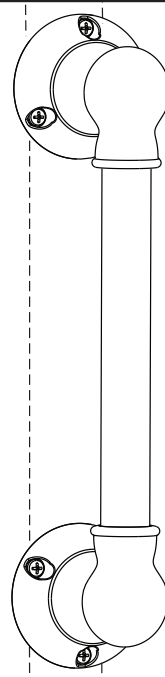WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 250 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact Moen immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with Moen's instructions. Do not modify this product in any way. Check grab bar periodically after installation to be sure all screws are secure. **FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT MOEN AT 1-800-289-6636.**

WARRANTY FOR MOEN HOME CARE BATH SAFETY PRODUCTS:**LIMITED LIFETIME WARRANTY**

Moen warrants to the original consumer purchaser for as long as the original consumer purchaser owns their home (the "Warranty Period" for homeowners) if this product fails due to materials or workmanship at any time during the life of the product Moen will replace it free of charge. Purchasers for residential use where the occupant is not the owner of the property (apartments, duplexes, or other multi-residential unit housing) are warranted for a period of 10 years from the original date of purchase (the "Warranty Period" for multifamily owners). Purchasers for industrial, institutional and all other business uses are warranted for a period of 5 years from the original date of purchase (the "Warranty Period" for commercial owners.) Proof of purchase (original sales receipt) from the original consumer purchaser must accompany all warranty claims. Contact Moen at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. **TO THE EXTENT PERMITTED BY LAW, MOEN DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.**

LIMITATION OF LIABILITY:

THIS WARRANTY IS YOUR SOLE REMEDY AND MOEN SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL MOEN'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.moen.com

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Moen Incorporated ©2019 • www.moen.com